



Technical Release Bulletin SQAIIC-P

Issue date: December 7, 2010

Subject: SQAIIC-P Printing Bug

Update:

When the SMI reading is in the range of 10-19, it is reported correctly on the device display, but not correctly in the printed test report. In the printout, the first digit (1) is missed. All other SMI ranges are both displayed and printed correctly.

Procedure:

- Upon completing the test and printing results (if required), compare the SMI printout with the result on the SQAIICP display.
- If different, correct the SMI printout manually based on the displayed result.

Compliance Date: Effective December 7, 2010

Authority: Lev Rabinovich, PhD CTO levr@mes-ltd.com

Distribution: All distributors; MES, LLC



www.mes-ltd.com Tel: 972-4-637-3981 Fax: 972-4-637-3984

Technical Release SQAIIC-P Printing Bug