

How to Load the I-button in SQA's with I-button Clips

Applies to the following: ALL SQA-V (V, Vb, Ve, Vp and Vt) with new i-button clip hardware

Issue date: February 24, 2010

Procedure for LOADING THE I-BUTTON into the Clip:

1. Turn on the SQA-V and the V-Sperm computer
2. From the MAIN MENU of the SQA-V select: SERVICE > ADD I-BUTTON TESTS
3. From V-Sperm (or B-Sperm, or E-Sperm, etc.) click: CONTINUE

4. Carefully remove the I-button from its card (see Fig. 2).
Please Note: Remove any glue from the button using an alcohol wipe.



Figure 2: Remove the I-button from its card

5. Insert the I-button into the i-button port of the SQA:
 - The smooth/larger side of the i-button should be facing up (See Fig. 4).
 - The i-button should be placed snugly into the port and held firmly in place by the clip (See Fig. 5).
 - There is no need to hold the I-button during the loading process – the clip will do this.

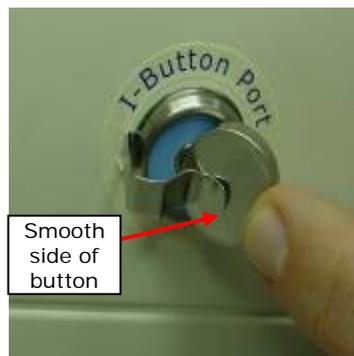


Figure 4: I-button insertion



Figure 5: Correct placement of the I-button in the port

6. Proceed to **V-Sperm** and click: **CONTINUE** (See Fig. 6).
7. A progress bar will be displayed on the V-Sperm screen as tests are transferred.
8. **Do not touch or remove the i-button** until this message appears: **I-Button Tests successfully updated** (see Fig. 7).



Figure 6: Click: CONTINUE from V-Sperm



Figure 7: Successful i-button loading!



www.mes-global.com



www.a-tech-global.com