Appendix IX: SQA-Vb System Specifications

Dimensions:40 x 30 x 15 cmWeight:4 kg

AC power supply: 100 to 250 VAC, 50/60 Hz, 10 VA

Measurement Compartment

- Sources of radiant energy two 880 nm LEDs for motility and spectrophotometry channels
- Detector system 2 photo detectors Motility and Optical Density

Visualization Compartment

- Green LED illumination system
- CCD, 350 TV lines
- Objective lens: Standard, x20
- Signal Output: PAL standard
- Zoom system for smooth magnification transition from x300 to x500
- Focus regulator

Display(s)

- Operational backlight LCD (16 lines x 40 characters)
- Video backlight LCD (8 x 10 cm)

Printer

- Built-in, Dot Matrix
- Non-thermostatic narrow paper with 20 characters per line (Citizen)
- Ribbon cassette (Citizen)

Keypad

- **Operational keys:** ON/OFF, TEST, PRINT, SERVICE, ARCHIVE, DELETE, ENTER, four cursor buttons, ESC, numeric buttons (0-9)
- Video control keys: ZOOM IN/OUT, ILLUMINATION HIGH/LOW, and MONITOR ON/OFF

Front Panel

- Built-in printer
- Visualization compartment
- LCD video display and controls
- Focus knob
- LCD operational display
- Measurement compartment
- Multi-button keypad

Rear/Side Panel

- Power connector with fuse-holder (fuse 250V, 1A)
- Video connector
- RS232 cable outlet
- I-Button port (side panel)

Specimen Testing Supplies

- **Measurement capillary:** Disposable, multi use plastic, positive displacement testing capillary (purchase from manufacturer).
- Standard lab slide: 76 X 25.6 mm with 22 x 22 mm cover-slip.
- I-Button: Required to run tests (purchase from manufacturer)

Archive Capacity

• 500 test records in each archive (Test and Controls)

Operating System

- Control: Keypad
- Analysis Time: Normal Test 50 seconds
- **Software:** Resides on flash memory and drives all man-machine interface functions, runs algorithms for test measurements and operational screens. System can be upgraded from a PC CD-ROM.
- **Sample Testing Temperature:** Calibrated for room temperature only. Motility results will be impacted by heating the specimen.
- Motility channel input signal: Analog, up to 5V.
- **Spectrophotometer channel input signal:** Modulated (1 kHz) analog, up to 5V.

Quality Control

- Internal: Electronic Self-Test and Auto-Calibration.
- External: Latex beads control material (QwikCheck[™] beads)

PC Compatibility

Minimum requirements for B-Sperm[™] software

- PC: 1 GHz processor, Pentium 3
- **RAM:** 256 MB
- AGP-video display card with at least 16 MB of RAM memory
- Video color: At least 16 bit (65,535)
- CD ROM drive
- 40 GB free hard disk space for image capturing/storage
- Video resolution: Minimum 640 x 480

Operating system compatibility

- Windows XP or VISTA
- **Ports:** One serial; two USB ports
- Monitor: 15" color

Additional Software (supplied with system)

• **B-Sperm software:** Real time visualization interface between PC and SQA-Vb visualization system, data transfer, video/picture capture and archive.

Operational Temperature and Humidity

- System is operational at 15-38°C.
- NOTE: SQA-Vb operates in a wide range of ambient temperatures however the system is calibrated to measure semen samples at room temperature: 22-26°C (68-79°F). Semen samples can be pre-heated to 37°C / 98.6°F prior to testing if required (refer to the SQA-Vb User Guide Appendix I: Semen Sample Preparation).
- *NOTE*: Variations in ambient temperature may impact the accuracy of test results because of the effect of temperature on semen.
- System is fully operational at up to 80% humidity and 31°C.

Maintenance Schedule

• Cleaning daily or after every 25 tests (refer to the SQA-Vb User Guide Appendix III: SQA-Vb Cleaning Instructions).

Manufacturer Recommendations

- Operate the SQA-Vb away from devices that may cause electronic noise (cell phones) or other devices causing vibrations such as centrifuges.
- Turn system **OFF** at the rear-panel when not in use for extended period of time.
- Variations in ambient temperature can affect semen samples. The SQA-Vb is calibrated to conduct tests at room temperature: 22-26°C (68-79°F). Nevertheless semen samples can be run by the system after pre-heating to 37°C / 98.6°F.
- Semen is considered a biologically hazardous material and is subject to individual laboratory protocols for handling such materials.

Factory Default Settings

Date format: **DD/MM/YY** Date/Time: Manufacturer's local date/time Sample Type: **FRESH** Fresh: **DOSING** Frozen Type: **STRAW**

Freezing Media Default Settings: **CLEAR** Operational Default Settings:

- Automatically Print: YES
- Display Test Results: **YES**
- Automatically Send to PC: YES
- Barcode Data Transfer: NO

Appendix X: Product Performance Data Abbreviations:

| CONC: | Sperm Concentration |
|-------|--|
| MSC: | Motile Sperm Concentration |
| PMSC: | Progressively Motile Sperm Concentration |
| Mic. | Micron |
| Sec. | Second |
| M/ml | Million per milliliter |

Performance Data Summary:

The performance of the SQA-Vb system for bull semen analysis is summarized in the text, tables and graphs below. Sperm concentration measurements are expressed as x10⁶ sperm cells per milliliter (M/ml). Motility and Morphology values are expressed as a percent (%). Unless otherwise noted all testing was performed using fresh and frozen bull semen samples.

Calibration:

Each SQA-Vb device is biologically calibrated against two reference systems at Medical Electronic System's laboratory using bull semen.

Dynamic Range:

| Sample Type | Test Mode | Conc. M/ml | Motility % | Morphology % | MSC M/ml | PMSC M/ml | Velocity, mic./sec. |
|----------------|--------------|---------------|---------------|--------------|-------------|--------------|------------------------|
| Fresh | Fresh | 0-2000 | 0-95 | 0-100 | 0-1900 | 0-1800 | 0-130 |
| Frozen | Frozen | 0-100 | 0-95 | - | 0-950 | 0-900 | 0-80 |

Precision and accuracy established against a known target (Latex beads)

Table 1: Precision

| known target (Latex beads) | SQA-V | Accu-beads® | CV % |
|--|--------------|-------------------|---------------|
| Background: The precision and accuracy of the | Intra-device | High 47±7.0 M/ml | ≤ 0.01 |
| SQA-V was compared to a known target value | Variability | Low 24 ± 3.4 M/ml | ≤ 0.01 |
| using commercially available latex beads of two | Inter-device | High 47±7.0 M/ml | ≤ 2.00 |
| concentrations. Latex beads are run on the SQA-V in the same manner as semen samples | Variability | Low 24 ± 3.4 M/ml | ≤ 2.50 |

Limitations of method:

Latex beads cannot:

Measure sperm motility or morphology

in the same manner as semen samples.

Correct for inaccurate chamber depths or technician errors


Method comparison:


A total of 320 latex bead samples were tested on ten SQA-V systems (32 samples per SQA-V). Precision of the SQA-V was estimated (Table 1). SQA-V concentration readings were compared to the established target values +/- the acceptable range for the latex beads (Fig. 1 & 2).

Accu-beads® published acceptable ranges (Hemacytometer):

- Vial #1: 47 +/- 7.0 M/ml
- Vial #2: 24 +/- 3.4 M/ml

Fig. 1-2 Accuracy Low/High Level Controls:

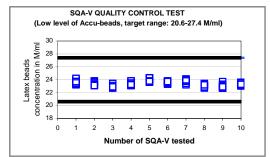

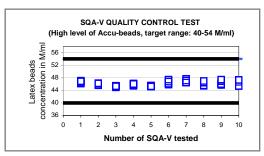

Conclusions:

The CONTROL mode software of the SQA-Vb (bull) device is exactly the same as the SQA-V (human) system. Both systems also have the same hardware platform. Therefore, the accuracy and precision results obtained on the CONTROL mode of the SQA-V will be the same as that of the CONTROL mode of the SQA-Vb.

Sensitivity, specificity, precision, accuracy and correlation to manual method established in the MES laboratory and field clinical trials using bull semen samples

FRESH SEMEN Performance Claims (Table 2-5)

Sensitivity

- Concentration: 90%
- Motility: 85%
- Morphology 80%

Specificity

- Concentration: 90%
- Motility: 80%
- Morphology 90%

Precision (CVs)

- Conc.: 3%
- Motility: 5%
- MSC: 7%
- Velocity: 10%

Accuracy (regression coefficients of "live/dead" trend line) Claims:

- Motility: 0.85
- MSC: 0.9
- PMSC: 0.9

Correlation to Manual Method Claims:

- Concentration: 0.9
- Motility: 0.8
- Morphology 0.7
- MSC: 0.9
- PMSC: 0.8
- Velocity: 0.75

FROZEN SEMEN Performance Claims (Table 3, 5)

Precision (CVs)

- MSC: 7%
- Velocity: 4%

Correlation to Manual Method

- MSC: 0.8
- PMSC: 0.7
- Velocity: 0.85

Table 2: Sensitivity/Specificity

| SQA-Vb vs. Microscope | Sensitivity | Specificity | |
|-----------------------------|-------------|-------------|--|
| FRESH SEMEN | | | |
| Sperm Concentration M/ml | 100.0% | 98.4% | |
| Motility, % | 96.3% | 85.0% | |
| Morphology | 85.7% | 91.7% | |

Table 3: Precision SQA-Vb intra-device Variability (CV, %)

| | Sample Type | | |
|-----------------------------|-------------|--------|--|
| Parameter | Fresh | Frozen | |
| Sperm Concentration M/ml | 2.4 | - | |
| Motility, % | 4.1 | - | |
| Morphology % | 5.0 | - | |
| MSC, M/ml | 6.1 | 6.4 | |
| Velocity, microns/sec. | 9.9 | 3.4 | |

Table 4: Accuracy: Regression coefficients from "live/dead" experiments

| Parameter | SQA-Vb | Manual |
|-------------|--------|--------|
| Motility, % | 0.9135 | 0.8325 |
| MSC, M/ml | 0.9348 | 0.7547 |
| PMSC, M/ml | 0.9340 | - |

Notes:

- Sensitivity and specificity claims are lower than actual values noted (Table 2).
- Precision CV claims are higher (lower precision) than actual values noted (Table 3).
- Accuracy regression coefficient claims are less than actual values noted (Table 4).
- Correlation to Manual Method claims are less than actual correlations noted (Table 5).

Method comparison:

SQA-Vb was compared to the microscope based on WHO '99 guidelines. The SQA-Vb automated readings of the sperm concentration, motility, MSC, PMSC and velocity were compared to microscopic results. A Makler chamber was used according to manufacturer's instructions for the manual sperm concentration measurements. A standard slide and B-Sperm software were used to assess manual motility, progressive motility and velocity measurements. Manual MSC and PMSC parameters were calculated from experimental results. The protocols were based on WHO '99 and MES guidelines. The alpha-site clinical trials were conducted at the Sion farm. A total of 104 fresh and 138 frozen semen samples were analyzed.

Analytical Specificity:

- To achieve analytical specificity a specific wave length of light which is maximally absorbed by sperm cells and minimally absorbed by other cells and seminal plasma is used.
- Low noise and high electronic resolution hardware components and compensation circuits ensure that analytical specificity is optimized.

Limitations of method:

Samples were assessed in duplicate on the automated SQA-Vb system and manually using a microscope. Statistical counting errors and intra-operator variability (subjectivity) may have affected the results of the study.

Accuracy assessment: "live/dead" sperm plots.

The SQA-Vb accuracy was assessed by "live/dead" bull sperm experiments. Fresh bull semen was distributed into two aliquots. The first aliquot was intact ("live") and the second one was treated with the liquid nitrogen ("dead"). Then different "live-to-dead" proportions were created providing constant а Sperm Concentration but varying MSC, PMSC and Motility. The samples were tested using the SQA-Vb device and under the microscope and the results plotted. The linear trend lines were established for motility, MSC and PMSC variables vs. "Live/Dead" sperm ratio.

Table 5: Correlation to manual method

| Parameters | Correlation coefficients | | | |
|---------------------------------|--------------------------|--------------|--|--|
| Farameters | Fresh semen | Frozen semen | | |
| Sperm Concentration, M/ml | 0.93 | - | | |
| Motility, % | 0.81 | - | | |
| Morphology % | 0.71 | - | | |
| MSC, M/ml | 0.94 | 0.84 | | |
| PMSC, M/ml | 0.86 | 0.74 | | |
| Velocity, mic./sec. | 0.81 | 0.91 | | |

Fig. 3: Method comparison: Regression plot of SQA-Vb Sperm Concentration in fresh bull semen vs. manual results

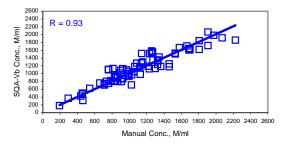


Fig. 4: Method comparison: Regression plot of SQA-Vb Motility in fresh bull semen vs. manual results


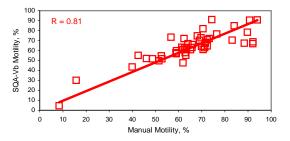


Fig. 5: Method comparison: Regression plot of SQA-Vb MSC in fresh bull semen vs. manual results

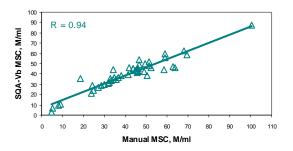

Performance parameters:

- Sensitivity and specificity were calculated using ROC analysis formulas. The cutoffs normally used for the sperm concentration and motility of the fresh semen samples were used for calculation of sensitivity and specificity. As for the frozen samples there are no cutoffs, sensitivity and specificity were not calculated for this type of samples (Table 2).
- Precision of the SQA-Vb device was estimated by calculation of the intra-device coefficients of variation (CV) of the duplicate measurements (Table 3). CV is calculated according to the formula: CV = SD / MEAN x 100
- The accuracy of the SQA-Vb device was characterized by regression coefficients of the trendline obtained in the "live/dead" experiment (Table 4).
- Correlation to manual method was established by calculation of correlation coefficients (Table 5, Fig. 3-5).

Conclusions:

- High levels of sensitivity, specificity and correlation of the SQA-Vb device results to the manual method were found. Therefore the instrument can be used in the field for the semen quality assessment, dose preparation and in the frozen semen QC.
- SQA-Vb provides the precise and accurate results with low coefficients of variation (<6%) and high regression coefficients of the "live/dead" trend lines (>0.95).