

Technical Release Bulletin
Self-Test Printout
Issue date: March 15th 2015

Background: To perform the SQA-V or QwikCheck GOLD Self-Test printout please follow the instructions below:

How to Print Self-Test Data (2 options)

Print Self-Test Data from the SQA or QwikCheck GOLD:

- Turn the SQA OFF and then ON again using the rear panel switch.
- Wait for stabilization, auto-calibration and self test to complete.
- From the **MAIN MENU**, select: **SERVICE MENU > PRINT DEFAULT SETTINGS** and press Enter; then highlight **SELF-TEST DATA** and press **ENTER**:

SERVICE MENU SERVICE DATA SERVICE PERSONNEL PRINT SQA-V DEFAULT SETTINGS ADD I-BUTTON TESTS	HIGHLIGHT OPTION AND PRESS ENTER TO PRINT 1. SELF-TEST DATA 2. CONTROLS / TEST SETTINGS
--------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------

- The **Self-Test Data** values will be printed.
- Send the results to MES for troubleshooting along with the other requested data.

Printing Self-Test Data from V-Sperm (SQA-V only):

- The SQA-V must be connected to the PC and V-Sperm activated.
- Turn OFF the device and ON again using the rear panel switch.
- Wait for stabilization, auto-calibration and self test to complete.
- From the **MAIN MENU** of the SQA-V select: **SERVICE > SERVICE DATA** screen of the SQA-V.
- Go to the V-Sperm navigation buttons and select: **UTILITIES > SELF TEST DATA**.
- Click **PRINT** and the Self-Test Data will be printed.

Compliance Date: Effective March 15th 2015 **Authority:** Dr. Lev Rabinovitch, CTO
Distribution: SQA-V and QwikCheck GOLD users

