



MEDICAL ELECTRONIC SYSTEMS
5757 W. Century Blvd. Suite 805
Los Angeles, CA. 90045
"Remember, it ALL Started with a Sperm!"
www.mes-global.com



SQA-VISION Semi-Annual Calibration Confirmation Instructions

Semiannual Instrument Calibration Confirmation

Twice per year, MES recommends sending us the "Service Data Report" of your SQA-VISION for a calibration confirmation. To print your Service Data Report and return it to MES, please follow these instructions:

- Right after powering the SQA-Vision "On" click on "REPORT" to the right of the home screen box.
- Print this report.
- Fax the report to MES (310-670-9069) or scan and e-mail it to service@mes-llc.com. Please include an e-mail address and your contact information with the Self-Test report so that we may return the confirmation.
- Your SQA-Vision calibration parameters will be compared to the initial calibration parameters recorded during the manufacturing process and presented back to you in report format