

SQA-Vision Fixed Cover-slip Slide

PRODUCT INSERT



INTRODUCTION AND INTENDED USE:

SQA-Vision Fixed Cover-slip Slides are used for automated and manual semen assessment. In the visualization compartments of the automated SQA Vision and SQA-V sperm analyzers the fixed cover-slip slide is recommended for the examination of oligospermic samples and to assess sperm concentration below the system's reportable range. The slide contains two chambers of a precise 20-micron depth. The advantage of using the SQA-Vision's Fixed Cover-slip slide is that slide calibration is not required and with the precise 20-micron depth chambers, filling errors are eliminated. The product is non-sterile and is intended for in vitro use only. Please note: There is no counting grid. Both the SQA-Vision and the SQA-V (V-Sperm) systems have grids for cell counting.

KIT CONTENTS:

- 24 double chamber slides per box and Product Insert

STORAGE, USAGE AND DISPOSAL CONDITIONS:

- Store at room temperature.
- Do not re-use, discard after use as a biologically hazardous waste material based on laboratory protocols.

INSTRUCTIONS FOR USE:

- Mix the semen sample thoroughly and pipette ~3.5 µl of semen.
- Load the sample where instructed by the arrow(s).
- After filling, place the slide into the slide-holder (SQA-Vision) or slide adaptor (SQA-V) or place under the microscope and assess the sample per the instructions in the user manuals (SQA-Vision and SQA-V) or per laboratory protocol.

