



SQA-VISION VERSION DIFFERENCES 3-27-17

Previous version	New version
SQA-Vision Testing device 3.00.60	SQA-Vision Testing device 3.00.61
SQA-Vision 44.3.167	SQA-Vision 44.3.180 and 44.3.182

- **Flexible Report** - A new custom (flexible) report feature was added. This allow the user to design their own report. MES has publish a guide on how to build out the Flexible report. Note: Programming Skills required.
- **Post Vas Counting Mode** - A new counting mode was added to the Post Vas. test. The new mode enable the user to scan an entire slide and click to count the cells without entering the number of fields and cells counted. The user will count all 50 fields available through the lab slide.
- **Multiple Users** - An option to add multiple users and password was added.
- **Range Icons** – Pass vs. Fail indication was replaced with up and down arrows based on the cutoff.
- **LIS Interface** - There is now an option to re-send a specific patient record as opposed to a whole date range. Additionally, an option to change the time out delay was added.
- **Time Format** - The time format is now set to a 24hour clock.
- **Debris Scanning Options** - A Gross Debris option was added to the debris scanner.

Previous version	New version
SQA-Vision Testing device 3.00.61	SQA-Vision Testing device 3.00.62
SQA-Vision 44.3.182	SQA-Vision 44.3.193

- **Back and Restore** – The process has been updated for ease of use and reliability.
- **Upgrade Process** – Streamlined and simplified for future versions.
- **Debris Scanner Workflow** – If the Low Quality Counter is turned **off** and the Debris Scanner is turned **on** the Debris Scanner will activate on Low Quality samples.
- **Stability Improvements** – The quality of data transfer and video interfacing is greatly improved for reliability.