

SQA-Vision Fixed Cover-slip Slide

PRODUCT INSERT

INTRODUCTION AND INTENDED USE:

SQA-Vision Fixed Cover-slip Slides are used for automated and manual semen assessment in the visualization system of the of the SQA Vision and SQA-V sperm analyzers. This fixed cover-slip slide is recommended for examination of oligospermic samples and to assess sperm concentration below the SQA reportable range with accuracy and without filling errors or slide calibration. Two chambers of 20-micron depth are located on each end of the slide for proper alignment with the SQA visualization system. The product is non-sterile and is intended for in vitro use only.

RESTRICTIONS: (1) No counting grid is provided on the slide - grids are provided on the SQA visualization screen (2) For use with sperm samples OR latex beads.

KIT CONTENTS:

- Product insert and 25 double chamber slides.

STORAGE, USAGE AND DISPOSAL CONDITIONS:

- Store at room temperature.
- Do not re-use, discard after use as a biologically hazardous waste material based on laboratory protocols.

INSTRUCTIONS FOR USE:

- Mix the semen sample thoroughly and pipette ~5 µl of semen.
- Load the sample where instructed by the arrow(s).
- After filling, place the slide into the slide-holder (SQA-Vision) or slide adaptor (SQA-V) or place under the microscope and assess the sample per the instructions in the user manuals (SQA-Vision and SQA-V) or per laboratory protocol.

CE



www.mes-global.com

Revision: 22_OCT_2022