



MEDICAL ELECTRONIC SYSTEMS  
5757 W. Century Blvd. Suite 805  
Los Angeles, CA. 90045  
"Remember, it ALL Started with a Sperm!"  
[www.mes-global.com](http://www.mes-global.com)



## SQA-V Gold Semiannual Calibration Confirmation Instructions

---

### OVERVIEW

---

It is recommended that twice per year, the SQA-V Gold calibration is checked against the original factory calibration parameters. Although there are acceptable calibration ranges for the SQA-V, the system parameters may be close to the high or low end of the range and proactive maintenance will ensure continued uninterrupted use and optimal clinical performance from the instrument.

---

### CALIBRATION CONFIRMATION INSTRUCTIONS

---

Twice per year, MES recommends sending us the "Service Data Report" of your SQA-V Gold for a calibration confirmation. To print your Service Data Report and return it to MES, please follow these instructions:

- NOTE: The SQA-V internal printer will need to be loaded with paper and printer ribbon. Please see user guide for paper and ribbon loading instructions.
- BEFORE Testing a Semen Sample or Running QC, Navigate to the "**Service**" menu. Once in the "**Service**" menu select "**Print Default Settings**" and click "**Print Self-Test Data**".
- Tape this report to a piece of blank white paper and fax the report to MES @ 310-670-9069 or scan and e-mail it to [service@mes-llc.com](mailto:service@mes-llc.com). Please include an e-mail address and your contact information with the Self-Test report.
- Your SQA-V calibration parameters will be compared to the initial calibration parameters from when the instrument was manufactured and presented back to you in report format